

QUICK START GUIDE

Get GEMS™ Mobile ECG







Setup your user and verify your email address

Connect & Record your ECG





App Install - GEMS™ Mobile ECG is available through the Google Play Store and the Apple App Store. Install the GEMS™ Mobile ECG App on your smart device and launch the application. Create a new user and verify your email address.

Connecting - Ensure Bluetooth is enabled on your smart device. Select Cannect to a Device button on the welcome screen; then select the HeartCheck™ CardiBeat from the list of devices and press the Cannect button. Hold the CardiBeat as shown to activate it.

Recording - GEMS™ Mobile ECG will begin recording after connecting to the device. The screen will show a countdown of 30 seconds while it records however an ECG trace will not be visible on first use. Add a diary entry at the end of recording (symptoms and activity), then upload for a free physician review to unlock ECG viewing.

Questions? Troubles? Get in touch with us at support@theheartcheck.com 1-877-977-9425 ex. 4 [+1-416-977-9425 ex 4. outside North America]

View more info on our website at: https://theheartcheck.com/cardibeat/